hyaluvisk drops

STERILE A

NON-INJECTABLE

Hydrating and lubricating solution, sterile, free from preservatives. Box of 20 x 0,5 ml multidose reclosable units in sachets.

COMPOSITION

Sodium hyaluronate 0,4%, dibasic sodium phosphate, monobasic sodium phosphate, sodium chloride, water for injections.

INDICATIONS


- Hyaluvisk drops is indicated in cases of eye dryness sensation when natural tear production is insufficient.
- Hyaluvisk drops plays an essential role in providing a stable and soft coating on the eye surface, which is maintained even with blinking. Hyaluvisk drops guarantees long lasting relief with maximum comfort.
- Hyaluvisk drops can be used for lubricating the eyes when the lids are not closed during general anaesthesia.

HOW DOES HYALUVISK DROPS WORK?

Hyaluvisk drops contain high molecular sodium hyaluronate, a natural polymer which is also present in eye tissues. Its water retention capacity plays an essential role in maintaining a film of tears. Its balanced formulation allows its pH and osmolarity to be adjusted to values similar to those of tears, and gives the product an irrigating, refreshing and soothing effect on sensitive eyes. The preservative-free formulation enhances the eye's excellent long-term tolerance, allowing regular, daily use.

HOW DO I USE HYALUVISK DROPS?

Dosing: place 1 or 2 drops into the conjunctival sac of the eye.

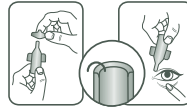Reclosable - Smooth tip for increased safety

Wash your hands thoroughly before using the solution. Tear off one multi-use container from the strip, then twist its top to open. Avoid touching the eyelid or eye surface with the container tip. The vial can be reclosed and once opened each vial can be used for up to 12 hours.

PRECAUTIONS

- Before use, make sure that the multi-use container is not damaged.
- Do not touch the eyelid or eye surface with the container tip.
- Once opened, each vial can be used for up to 12 hours.
- Does not contain any preservative.
- Do not use after the expiry date indicated on the package.
- Store below 25°C.
- After opening of the sachet, keep the multidose units safe from the light.
- Keep out of the reach of children.
- In case of concomitant treatment, do not apply Hyaluvisk drops at the same time as other eye drops.
- If discomfort persists while using Hyaluvisk drops or reappears after discontinuation of treatment consult your doctor.

NTC srl, 20124 Milaan, Italie

Distributed by: ID PHAR, Iddergemse Steenweg 77, BE-9450 Haaltert

Le Hyaluvisk drops is a classe IS medical device

Date of the first EC certificate: September 2011

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